



AeroText™ for Android™ Bluetooth® Calling and Texting Quick Reference Guide for NuVoice® Software ver. 2.15

PRC Tech Support: (800) 262-1990

service@prentrom.com

www.prentrom.com

STEP 1: DOWNLOAD AND INSTALL THE AEROTEXT APP ONTO YOUR ANDROID PHONE

Texting requires an Android phone with the PRC-Salttillo AeroText app installed. To install the app, type the following address into the browser on your phone or point your phone's camera at the QR code below.

<https://prc-salttillo.com/texting/>



Important! It is critical that you scroll down and read all of the directions for installing the app on your phone before checking the End User License Agreement box.

Follow the prompts to install the app on your phone. Be sure to set the AeroText app as your default messaging app and make sure Bluetooth is enabled and your phone is discoverable.

STEP 2: PAIR YOUR PHONE TO THE ACCENT® DEVICE

1. Select **TOOLBOX**.
2. Select **MAINT MENU**.
3. Select **HARDWARE DIAGNOSTICS**.
4. Select **PHONE CONFIGURATION**.
5. Select **PAIR TO PHONE**.
6. Enter **0000** on your phone if prompted for a PIN.
7. Bluetooth status on your Accent should display **Connected**. Then select **OK** to exit out of the toolbox.

STEP 3: SET UP FOR CALLING AND TEXTING

1. From the PHONE CONFIGURATION menu, set *USE PHONE FOR* to **Calling and Texting**.
2. Select **LOAD OR UPDATE PHONE PAGES** and follow the prompts.

Note: If *USE PHONE FOR* is set to **CALLING** only, you will not be able to send/receive texts or be able to navigate to the **PHONE TEXTING PAGE**.



MAKING A CALL

1. Navigate to the **PHONE/CALLING** pages in your vocabulary.
2. Select the **CALL** tab.
3. Select the **CONTACT** you would like to call. If there are no contacts listed, select **MANUAL DIAL** to dial the phone number directly and press **SEND**.
4. After selecting the **CONTACT** you will be taken to the **PHONE ON A CALL PAGE**. Use the **KEYBOARD** or **QUICK MESSAGES** from this page or your vocabulary to talk on the phone.
5. To end the call, select the red **END CALL** key.

Texting

1. Navigate to the **TEXTING** pages in your vocabulary.
2. Select the **ANDROID TEXTING** tab.
3. Select **NEW CONVO** to navigate to the **PHONE TEXTING CONTACTS** page to start a new conversation.
4. Read existing conversations by selecting one of the yellow conversation keys. Details of the conversation will appear on the blue message keys.
5. To copy, view, or reply to a text message, first select it from the blue column. Then select the command from the white column.



Helpful Tips

The following options are available on the **PHONE SETTINGS PAGE**. Turn the **PHONE MIC ON** if you want others to hear your natural voice through the phone as well as the speech generated from the Accent device.

