


Turn up the volume and be heard.

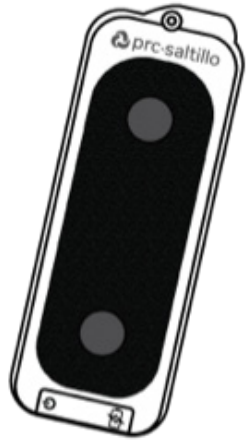

VERSA SPEAKER™


USER GUIDE

Get Started

Turning Power On/Off

- To Turn on Press I/O button. You will hear a tone that indicates the speaker is powered on. Speaker will automatically go into pairing mode.
- To Turn off Press & hold I/O button for 7 seconds.
- To Hard Reset, when necessary, press and hold I/O button for more than 10 seconds.


I/O Button

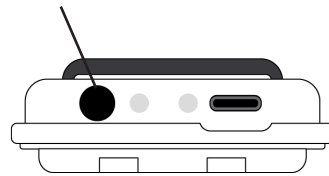


Connect to Device

Pairing VersaSpeaker with iPad

1. Hold I/O button down for 3 seconds.
2. Flashing blue light indicates pairing mode.
3. Solid blue light indicates the speaker has successfully paired.

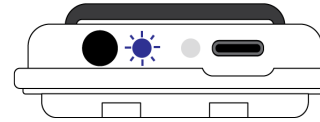


Pairing mode

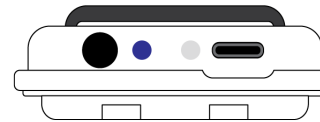

Successfully paired

Charge Your Speaker

1. Plug charging cable into USB C Charger Port.
2. The indicator light should be flashing orange to indicate it is charging.
3. Solid orange light indicates the battery is charged (when charger attached).

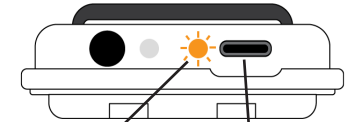


Indicator Light

USB C Charger Port

Note: *Low Battery* When orange light flashes twice every 30 seconds battery is below 15%. Typical battery life is 9 hours.


Attach the VersaSpeaker to the VersaWrap

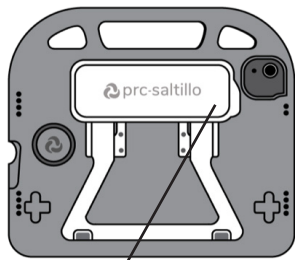

Speaker Cover Plate

Removing Cover Plate


1. Grab the tab on the right side of the cover plate.
2. Lift to the plate to remove.

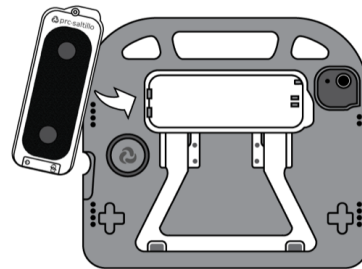

Attaching Speaker

1. With cover removed, align tabs on left side and slide speaker into place.
2. Then, on the right side, press down until speaker clicks in.
3. Device can be secured with screw on the right side, if desired.


Screw location

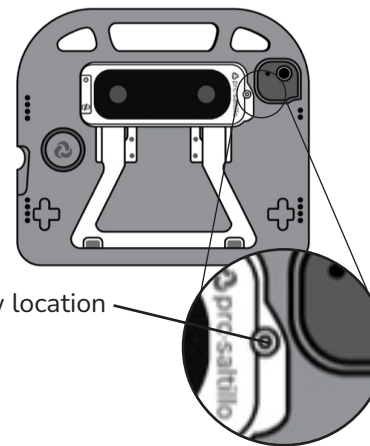

Contact us for support at versasupport@prc-salttillo.com or visit prc-salttillo.com/versa

Light Indicators Reference

| LIGHT | STATUS |
|------------------------------|-----------------------------|
| Flashing Blue | Pair Mode |
| Solid Blue | Paired |
| Flashing Orange | Charging (while plugged in) |
| Solid Orange | Charged |
| Intermittent Flashing Orange | Battery level below 15% |

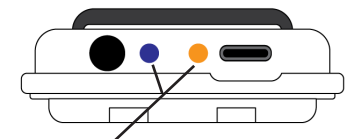

Indicator Lights